# CanAg SCC EIA

REF 800-10

IVD

Instrucciones de uso. 2009-11

EN	EXPLANATION OF SYMBOLS
BG	ОБЯСНЕНИЕ НА СИМВОЛИТЕ
CS	VÝZNAM SYMBOLŮ
DA	SYMBOLFORKLARING
DE	ERKLÄRUNG DER SYMBOLE
EL	ΕΠΕΞΗΓΗΣΗ ΤΩΝ ΣΥΜΒΟΛΩΝ
ES	SIGNIFICADO DE LOS SÍMBOLOS
ET	SÜMBOLITE SELGITUS
FR	EXPLICATION DES SYMBOLES
HR	OBJAŠNJENJE SIMBOLA
HU	JELMAGYARÁZAT
IT	SPIEGAZIONE DEI SIMBOLI
LT	SIMBOLIŲ PAAIŠKINIMAI
LV	SIMBOLU SKAIDROJUMS
NL	VERKLARING DER SYMBOLEN
NO	SYMBOLFORKLARING
PL	OBJAŚNIENIE SYMBOLI
PT	EXPLICAÇÃO DOS SÍMBOLOS
RO	SEMNIIFICAȚIA SIMBOLURILOR
RU	ОБОЗНАЧЕНИЯ
SE	SYMBOLFÖRKLARING
SK	VÝZNAM SYMBOLOV
SL	RAZLAGA SIMBOLOV
SR	OBJAŠNJENJE SIMBOLA
TR	SEMBOLLERİN AÇIKLAMALARI

Use By/Годно до/Použitelné do/  
Holdbar til/Verwendbar bis/  
Ημερομηνία λήξης/Fecha  
de caducidad/Kölblik kuni/  
Utiliser jusque/Rok valjanosti/  
Felhasználható/Utilizzare entro/  
Sunaudoti iki/Izlietot līdz/Houdbaar  
tot/Brukes innen/Użyty przed/  
Prazo de validade/Expiră la/  
Использовать до/Använd före/  
Použite'né do/ Uporabno do/  
Upotrebljivo do/Son Kullanna Tarihi

LOT

Batch code/Номер на партида/  
Číslo šarže/Lotnummer/  
Chargenbezeichnung/Αριθμός  
Παρτίδας/Código de lote/Partii  
kood/Code du lot/Kod serije/  
Sarzszzám/Codice del lotto/  
Partijos kodas/Partijas kods/Lot  
nummer/Partikode/Kod partii/  
Código do lote/Număr de lot/  
Номер лота/Lotnummer/Číslo  
šarže/Številka serije/Kod partije/  
Parti Kodu

Date of manufacture/Дата на производство/Datum výroby/  
Produktionsdato/Herstellungsdatum/  
Ημερομηνία παραγωγής/Fecha de fabricación/Valmistamise kuupäev/  
Date de fabrication/Datum proizvodnje/  
Gyártási idő/Data di produzione/  
Pagaminimo data/Ražošanas datums/  
Productiedatum/Fremstillingsdato/  
Data produkci/Data de fabrico/Data fabricației/Дата производства/  
Tillverkningsdatum/Dátum výroby/Datum izdelave/Datum proizvodnje/Üretim tarihi

In Vitro Diagnostic Medical Device/  
Медицински уред за диагностика  
ин витро/Lékařský přístroj pro diagnostiku in vitro/Medicinsk udstyr til in vitro-diagnostik/In-vitro-Diagnostikum/  
Ιατροτεχνολογικό προϊόν για διάγνωση  
In Vitro/Dispositivo médico para diagnóstico in vitro/In vitro diagnostiline meditsiiniseade/Dispositif médical de diagnostic in vitro/Diagnostički medicinski uređaj In Vitro/In vitro orvosdiagnostikai eszköz/Dispositivo medico per test diagnostici in vitro/In Vitro Diagnostiné Medicinos Priemonė/Medicīniska ierīce in vitro diagnostikai/  
In vitro-diagnostisch medisch instrument/  
In vitro diagnostisk medisinsk utstyr/  
Wyrób medyczny do diagnostyki in vitro/  
Dispositivo Médico de Diagnóstico In Vitro/Dispozitiv medical pentru diagnostic in vitro/Только для диагностики In Vitro/Endast för in vitro-diagnostik/  
Zdravotnícka pomôcka na diagnostiku in vitro/In vitro diagnostični pripomoček/  
Diagnostički medicinski uređaj In Vitro/<96> testleri için yeterlilik içerir

Catalogue number/Каталожен номер/  
Katalogové číslo/Katalognummer/  
Bestellnummer/Αριθμός καταλόγου/  
Número de catálogo/Kataloogi number/  
Numéro de catalogue/Kataloški broj/  
Katalógusszám/Numero di catalogo/  
Katalogo numeris/Numurs katalogā/  
Catalogusnummer/Katalognummer/  
Numer katalogowy/Número do catálogo/  
Număr de catalog/Номер по каталогу/  
Produktnummer/Katalógové číslo/  
Kataloška številka/Kataloški broj/  
Katalog numarası

Temperature limitation/  
Температурни граници/  
Teplotní omezení/  
Temperaturbegrænsning/  
Temperaturbegrenzung/  
Περιορισμοί θερμοκρασίας/  
Limites de temperatura/  
Temperatuuri piirang/  
Limite de température/  
Temperaturno ograničenje/  
Hőmérsékletre vonatkozó korlátozás/  
Limiti di temperatura/  
Temperatūriniai apribojimai/  
Temperatūras ierobežojums/  
Temperatūrbepērkings/  
Temperaturbegrensninger/  
Temperatury graniczne/  
Limite de temperatura/  
Limite de temperatură/  
Температурный режим/  
Temperaturbegrænsning/  
Teplotné obmedzenie  
Omejitve temperature/  
Temperaturno ograničenje/  
Sıcaklık sınırlaması/

Contains sufficient for <96> tests/Συдържа достатъчно количество за тестове <96>/Lze použít pro <96> testů/Ineholder tilstrækkeligt/Inhalt ausreichend für <96> Prüfungen/Περιεχόμενο επαρκές για «96» εξετάσεις/Contenido suficiente para <96> ensayos/Kogusest piisab <96> testi läbiviimiseks/Contenu suffisant pour «96» tests/Sadrži dovoljno za <96> testova/A doboz tartalma <96> vizsgálat elvégzéséhez elegendő/Contenuto sufficiente per «96» saggi/Turinys skirtas atlikti <96> tyrimus/Saturs pietiekams <96> testiem/Inhoud voldoende voor «96» testen/til «96» test/Tilstrekkelig innhold for <96> prøver/Wystarczy na wykonanie <96> testów/Conteúdo suficiente para «96» ensaios/Conținut suficient pentru 96 de teste/Содержит достаточные количества для «96» определений/Innehåller tillräckligt till «96» antal tester/Obsah postačuje na tento počet testov: <96>/Vsebinsa zadostuje za <96> testov/Sadržina dovoljna za <96> testova/<96> testleri için yeterlilik içerir

Consult Instructions for Use/  
 Прочетете инструкцията за  
 употреба/Konzultujte s návodem  
 k použití/Se brugsanvisning/Siehe  
 Gebrauchsanweisung/Συμβουλευτείτε  
 τις Οδηγίες σχετικά με τη χρήση/  
 Consulte las instrucciones de uso/  
 Vt kasutusjuhendit/Consulter le mode  
 d'emploi/Pročítajte upute za uporabu/  
 Olvassa el a használati utasítást/  
 Consultare le istruzioni per l'uso/Dël  
 naudojimo žiūrėkite instrukcijas/Izlasiet  
 lietošanas instrukciju/Raadpleeg de  
 instructies voor gebruik/Les instruksene  
 før bruk/Sprawdzić w instrukcji użycia/  
 Consulte as Instruções de Utilização/  
 Consultați instrucțiunile de utilizare/  
 Обратитесь к инструкции по  
 применению/Se bruksanvisning/  
 Prečítajte si návod na používanie/  
 Pročítajte uputstvo za upotrebu/  
 Kullanım Talimatlarına Bakınız

## CONT

Contents of kit/Съдържание на набора/  
 Obsah sady/Kittets indhold/Inhalt des  
 Kits/Περιεχόμενα του κιτ/Contenido  
 del kit/Komplekt sisaldab/Contenu du  
 kit/Sadržaj opreme/A készlet tartalma/  
 Contenuto del kit/Rinkinio turinys/  
 Komplekta saturs/Inhoud van de set/  
 Settets innhold/Zawartość zestawu/  
 Conteúdo do kit/Conținutul setului/  
 Компоненты набора/Kit innehåll/  
 Obsah súpravy/Vsebina kompleta/Sadržaj  
 opreme/Kitin içindekiler

Biological risks/Биологическа  
 опасност/Biologická rizika/Biologisk  
 fare/Biologische Gefahren/Βιολογικοί  
 κίνδυνοι/Riesgos biológicos/  
 Bioloogilised ohud/Risques biologiques/  
 Biološkli rizici/Biológiai kockázatok/Rischi  
 biologici/Biologinis pavojus/Bioloģiskais  
 risks/Biologische risico's/Biologiske  
 risikoer/Zagrozenie biologiczne/Riscos  
 biológicos/ Biologisk risk/Pericole  
 biologice/Биологическая опасность/  
 Biologicky rizikové/Biologické riziká/  
 Biološkli rizici/Biyolojik riskler

## ORIG HUM

Human/С човешки произход/Lidské/  
 Humant/Human/δείγματα αναφοράς/  
 Humano/Inimpăritolu/Humaine/Ljudskog  
 porjekla/Humán/Origine Umana/  
 Žmogaus kilmės/Cilvēku izcelsmes/  
 Human/Menneske/Ludzka/Humano/  
 Origine umaná/Человеческого  
 происхождения/Human/Ludské/  
 Humanega izvora/Ljudskog porekla/İnsan

## ORIG MOU

From mouse/С миши произход/Мыši/  
 Fra mus/der Maus/από ποντίκι/de ratón/  
 Hiirtelt/De souris/Mišijeg porjekla/  
 Egérbóli/Murino/Pelés kilmės/No peles/  
 Van muizen/Fra mus/Mysia/Do rato/De  
 la șoareci/Мышиного происхождения/  
 Från mus/Myšie/Mišjega izvora/Mišijeg  
 porekla/Fareden

## ORIG BOV

Bovine/С говежди произход/  
 Hovězí/Bovin/Rind/από βοοειδή/  
 Bovino/Veistelt/Bovine/Rogate stoke/  
 Szarvasmarha/Bovino/Jaučio/No  
 liellopa/Bovien/Bovin/Wołowy/Bovino/  
 Origine bovină/крупного рогатого  
 скота/Från ko/Hovädzie/Govejega  
 izvora/Rogate krupne stoke/Bovin

Reconstitute with/Разтваряне с/  
 Rozřed'te pomocí/Rekonstitueres med/  
 Rekonstituieren mit/Ανασύσταση με/  
 Reconstituir con/Lahjendamine/  
 Reconstituer avec/Rekonstituiraite s/  
 Feloldáshoz/Ricostituire con/LT/Atšķaidīt  
 ar/Reconstitutie met/Rekonstitueres  
 med/Odtworzyć za pomocą/Reconstituir  
 com/A se reconstitui cu/Растворить в/  
 Rekonstituera med/Rozried'te pomocou/  
 Rekonstituiraite z/s/Ponovno formiranje  
 sa/Yeniden oluřturulur

Manufacturer/Производитель/Výrobce/  
 Producent/Hersteller/Κτασκευαστής/  
 Fabricante/Tootja/Fabricant/Proizvođač/  
 Gyártó/Fabbricante/Gamintojas/  
 Ražotājs/Fabrikant/Produsent/  
 Producent/Fabricante/Producător/  
 Производитель/Тilverkare/ Výrobca/  
 Izdelovalec/Proizvođač/Üretici

# CanAg SCC EIA

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1. Suminami Y., Kishi F., Sekiguchi K., Kato H. (1991) Squamous cell carcinoma antigen is a new member of the serine protease inhibitors. *Biochem Biophys Res Commun* 181, 51-58.
2. Kato H. and Torigoe T. (1977) Radioimmunoassay for tumor antigen of human cervical squamous cell carcinoma. *Cancer* 40, 1621-1628.
3. Schneider S.S., Schick C., Fish K.E., Miller E., Pena J.C., Treter S.D., Hui S.M., Silverman G.A. (1995) A serine protease inhibitor locus at 18q21.3 contains a tandem duplication of the human squamous cell carcinoma antigen gene. *Proc Natl Acad Sci USA*, 92, 3147-3151.
4. de Bruijn H.W.A., Duk J.M., van der Zee A.G.J., Pras E., Willemse P.H.B., Boonstra H., Hollema H., Mourits M.J.E., de Vries E.G.E., Aalders J.G. (1998) The Clinical Value of Squamous cell Carcinoma Antigen in Cancer of the Uterine Cervix. *Tumor Biol* 19, 505-516.
5. Vassiliakopoulos T., Troupis T., Sotiropoulou C., Zacharatos P., Katsaounou P., Parthenis D., Noussia O., Troupis G., Papiris S., Kittas C., Roussos C., Zakynthinos S., Gorgoulis V. (2001) Diagnostic and prognostic significance of squamous cell carcinoma antigen in non-small cell lung cancer *Lung Cancer* 32, 137-144.
6. Snyderman C.H., Dámico F., Wagner R., Eibling D.E. (1995) A Reappraisal of the Squamous Cell Carcinoma Antigen as a Tumor Marker in Head and Neck Cancer. *Arch Otolaryngol Head Neck Surg* 121, 1294-1297.
7. Duk J.M., Groenier K.H., de Bruijn H.W.A., Hollema H., ten Hoor K.A., van der Zee A.G.J., Aalders J.G. (1996) Pretreatment Serum Squamous cell Carcinoma Antigen: A Newly Identified Prognostic Factor in Early-Stage Cervical Carcinoma. *J Clin Oncol* 14, 111-118. 1997
8. Fleisher M. et al. (2002) Practice guidelines and recommendations for use of tumor markers in the clinic. *National Academy of Clinical Biochemistry* 15: p.19.
9. Yuyama N., et al., (2002) Analysis of novel disease-related genes in bronchial asthma. *Cytokine* 19(6), 287-296.
10. National Committee for Clinical Laboratory Standards, Evaluation of Precision Performance of Clinical Chemistry Devices. Approved Guideline EP5-A (1999).
11. Röijer E., Nilsson K., Oskarsson M., Dahlén U., Andersson I., Nilsson O. (2003) Development of Monoclonal Antibodies and Immunoassays against different forms of Squamous Cell Carcinoma Antigens (SCCA). *Tumor Biol* 24, p. 83.
12. National Committee for Clinical Laboratory Standards, National Evaluation Protocols for Interference Testing, Evaluation protocol Number 7, Vol. 6, No 13, August (1986).

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CanAg® es una marca comercial registrada de Fujirebio Diagnostics AB
Fujirebio Diagnostics AB